




GAFCHROMIC® EBT-EASEL User Guide


The GAFCHROMIC EBT-EASEL is designed to help ensure correct film placement and flatness during the scanning of GAFCHROMIC EBT2 films, of size 8"x10", with the EPSON Expression 10000 XL Photo scanners.

To use the GAFCHROMIC EBT-EASEL :

- 1) Attach the EPSON Transparency Unit and the EPSON Transparency Guide if they are not already attached by following the instructions on the EPSON Expression 10000 XL User's Guide.
- 2) Place the GAFCHROMIC EBT-EASEL on the scanner bed aligned with the EPSON Transparency Guide as shown in the photograph below:

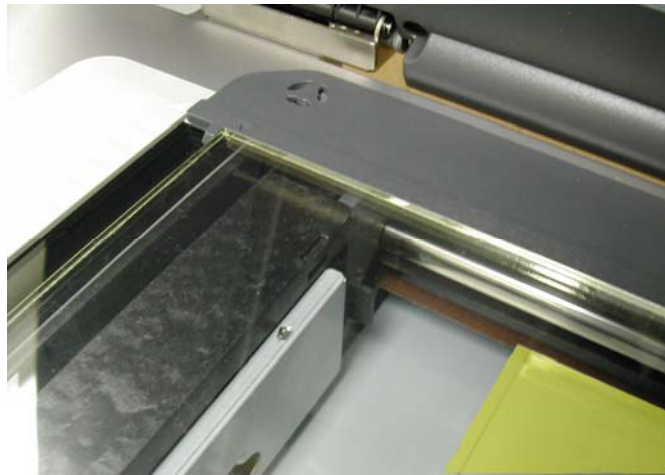

- 3) Place the GAFCHROMIC EBT2 film on the scanner bed in landscape orientation under the GAFCHROMIC EBT-EASEL as shown in the photograph below.

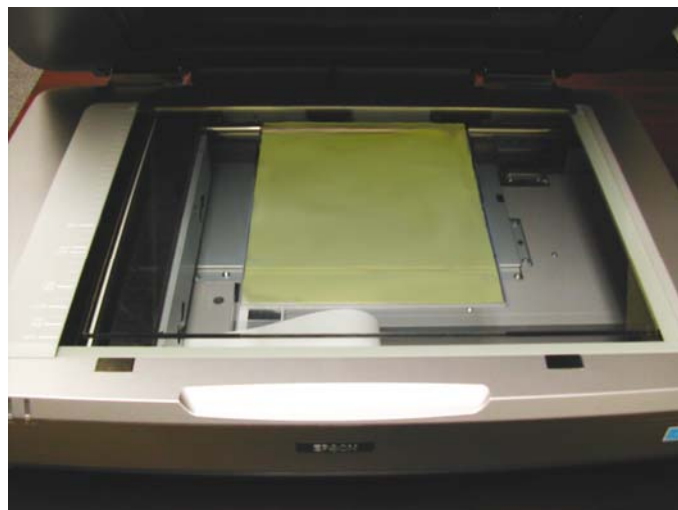

The cutout of the easel is slightly smaller than 8"x10" to push the edges of the film down. Thus a film that has a curl should be placed down with the edges up.

GAFCHROMIC is registered trademark of the ISP group. EPSON and EPSON Expression are registered trademarks of SEIKO EPSON Corporation.
© 2009 International Specialty Products. Designed & Printed in USA.

The information contained in this brochure and the various products described are intended for use only by persons having technical skill and at their own discretion and risk after they have performed necessary technical investigations, tests and evaluations of the products and their uses. While the information herein is believed to be reliable, we do not guarantee its accuracy and a purchaser must make its own determination of a product's suitability for purchaser's use, for the protection of the environment, and for the health and safety of its employees and the purchasers of its products. Neither ISP nor its affiliates shall be responsible for the use of this information, or of any product, method, formulation, or apparatus described in this brochure. Nothing herein waives any of ISP's or its affiliates' conditions of sale, and WE MAKE NO WARRANTY, EXPRESS OR IMPLIED, OF MERCHANTABILITY OR FITNESS OF ANY PRODUCT FOR A PARTICULAR USE OR PURPOSE. We also make no warranty against infringement of any patents by reason of purchaser's use of any information, product, method or apparatus described in this brochure.

